




- **NOTE:** This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device. Modifications not approved by the party responsible for compliance could void user's authority to operate the equipment.

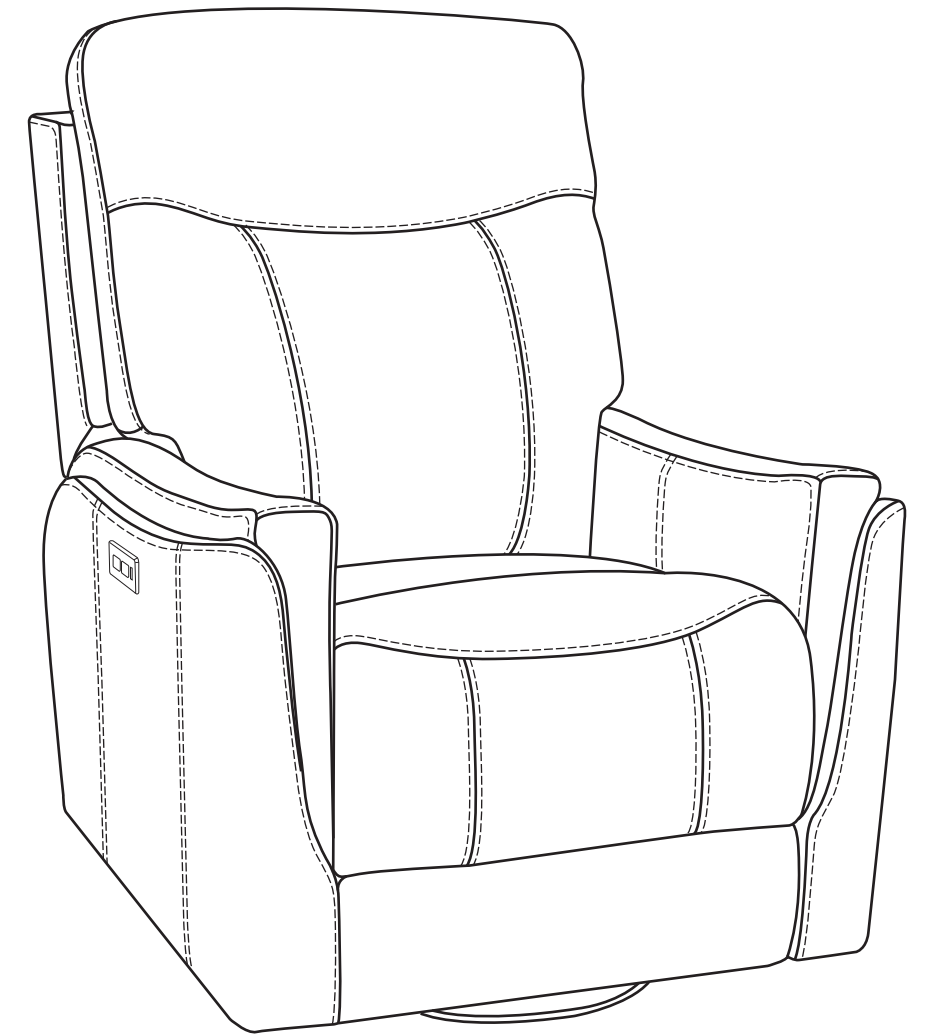

Setup and Operation Guide

Layflat Recliner

Model: B031T-006-874

TABLE OF CONTENTS

Important Safety Notice 3

Fabric Care 4

Assembly Of Your Lift Chair

Transformer and Cord Connections 4

Chair Assembly..... 5

Operating Your Lift Chair

Reclining the Chair Back Up or Down 6

Raising and Lowering the Chair 6

Lifting or Lowering the Legrest..... 6

Trouble Shooting..... 7

Note:


When placing the furniture on non-carpeted flooring, HMI recommend the use of floor protection such as an area rug or furniture pads. Area rugs and padding not included with purchase.

- **NOTE:** This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device. Modifications not approved by the party responsible for compliance could void user's authority to operate the equipment.



TROUBLE SHOOTING

PROBLEM	POSSIBLE FACTORS	SOLUTIONS
Back will not recline up or down	• Power supply is not connected completely to socket • Wiring plugs are not connected correctly according to the figure on page 4	• Check the power supply has been correctly connected to the socket • Confirm that wiring plugs are connected correctly
Legrest will not move up or down	• Power supply is not connected completely to socket • Wiring plugs are not connected correctly according to the figure on page 4	• Check the power supply has been correctly connected to the socket • Confirm that wiring plugs are connected correctly
Chair doesn't recline or lift	• Transformer may not be connected well	• Check the power supply has been correctly connected to the socket • Confirm the wire from the chair to the transformer is firmly connected.
		• If after making sure all connections have been properly made and the functions of the chair still do not perform, trying unplugging the chair from the outlet for ten minutes. Then replug the transformer into the power outlet and try again.

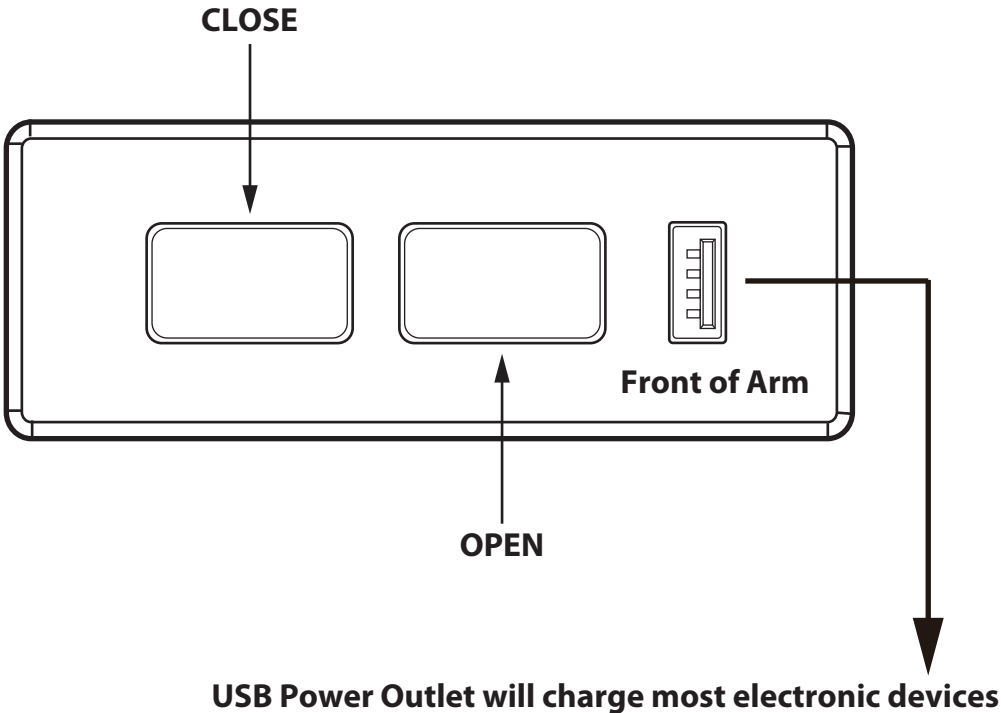

Care should always be taken to assure there is nothing in the path of the mechanism before and during operation.

Open and Close Buttons

The control panel uses rocker switches. To operate the function you desire simply touch the end of the rocker switch with the noted function. To stop the mechanism /function in your desired position simply remove your finger from the button.

***WARNING: Continuing to hold the Open Arrow will lift the chair.**

To Close All: To simultaneously return the headrest, recliner, and lumbar support to their closed positions, press and hold the end of rocker switch noted as mechanism close. To stop this function at any time simply remove your finger from the button.

***WARNING: Continuing to hold the close Arrow will raise the Footrest.**

To use the hidden cup holder: Push the front of the arm to gain access to the hidden cup holder.



IMPORTANT SAFETY INFORMATION

READ ALL INSTRUCTIONS BEFORE USING YOUR LIFT CHAIR.

WARNING: To reduce the risk of burns, fire, electric shock, or injury to persons:

1. **DANGER:** To reduce the risk of electric shock - Always unplug this furnishing from the electrical outlet before cleaning.
2. **Do not stand directly in front of chair when lowering the lifted chair to the seated position, continuing to hold the down arrow will allow the footrest to raise potentially causing the user to lose their footing.**
3. Unplug from outlet before putting on or taking off parts.
4. Close supervision is necessary when used by or near children, or disabled people.
5. Use this furnishing only for its intended use as described in these instructions. Do not use attachments not recommended by the manufacturer.
6. Never operate this furnishing if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped in water.
7. Never operate the furnishing with the air openings blocked. Keep the air openings free of lint, hair, and other obstacles.
8. Keep the cord away from heated surfaces.
9. Never drop or insert any object into any opening.
10. Do not use outdoors. Just for household indoor use only.
11. Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
12. To disconnect, turn all controls to the off position, then remove plug from outlet.
13. **WARNING: Risk of injury - Keep children away from extended foot support.**
14. **WARNING: Risk of injury to Persons - do not use this furnishing to support video equipment such as televisions or computer monitors.**
15. Maximum weight limit not to exceed more than 350 pounds.
16. When not in use, it is recommended that the wand be stored securely in the pocket on the outside of the arm.
17. **WARNING: Continuing to hold the up Arrow will lift the chair.**
18. **WARNING: Continuing to hold the down Arrow will raise the Footrest**
19. For loading always put heavier items at the bottom and not near the top in order to help prevent the possibility of the furnishing tipping over.

SAVE THESE INSTRUCTIONS.

SERVICING OF DOUBLE-INSULATED PRODUCTS.

In a double-insulated product, two systems of insulation are provided instead of grounding. No grounding means is provided on a double-insulated product, nor is a means for grounding to be added to the product. Servicing a double-insulated product requires extreme care and knowledge of the system, and is to be done only by qualified service personnel. Replacement parts for a double-insulated product must be identical to the parts they replace.


OPERATING YOUR LIFT CHAIR

The micro suede fabric covering this specific piece of furniture is soft to the touch and very durable for every day living. It is recommended by the fabric manufacturer to only clean this type of fabric with water-based cleaning agents or foam. Lightly scrubbing the fabric will help with any tougher spots that may have developed. Always check a spot of fabric in an area that is unseen to be sure the cleaning agent being used will not discolor or fade the fabric's finish. Let air dry and vacuum for best results.

CORD AND TRANSFORMER CONNECTIONS


Once the box has been removed from around the chair, you'll notice the back is laying on top of the seat. There are a couple items to remove from the seat portion of the chair first. Follow the steps below:

- 1. Remove the zip tie holding the white box with the Power Transformer and it's cord on the left as shown in the figure below.
- 2. Connect the Power Transformer to the plug coming from the back of the chair.
- 3. Connect the Power Cord to the Power Transformer.

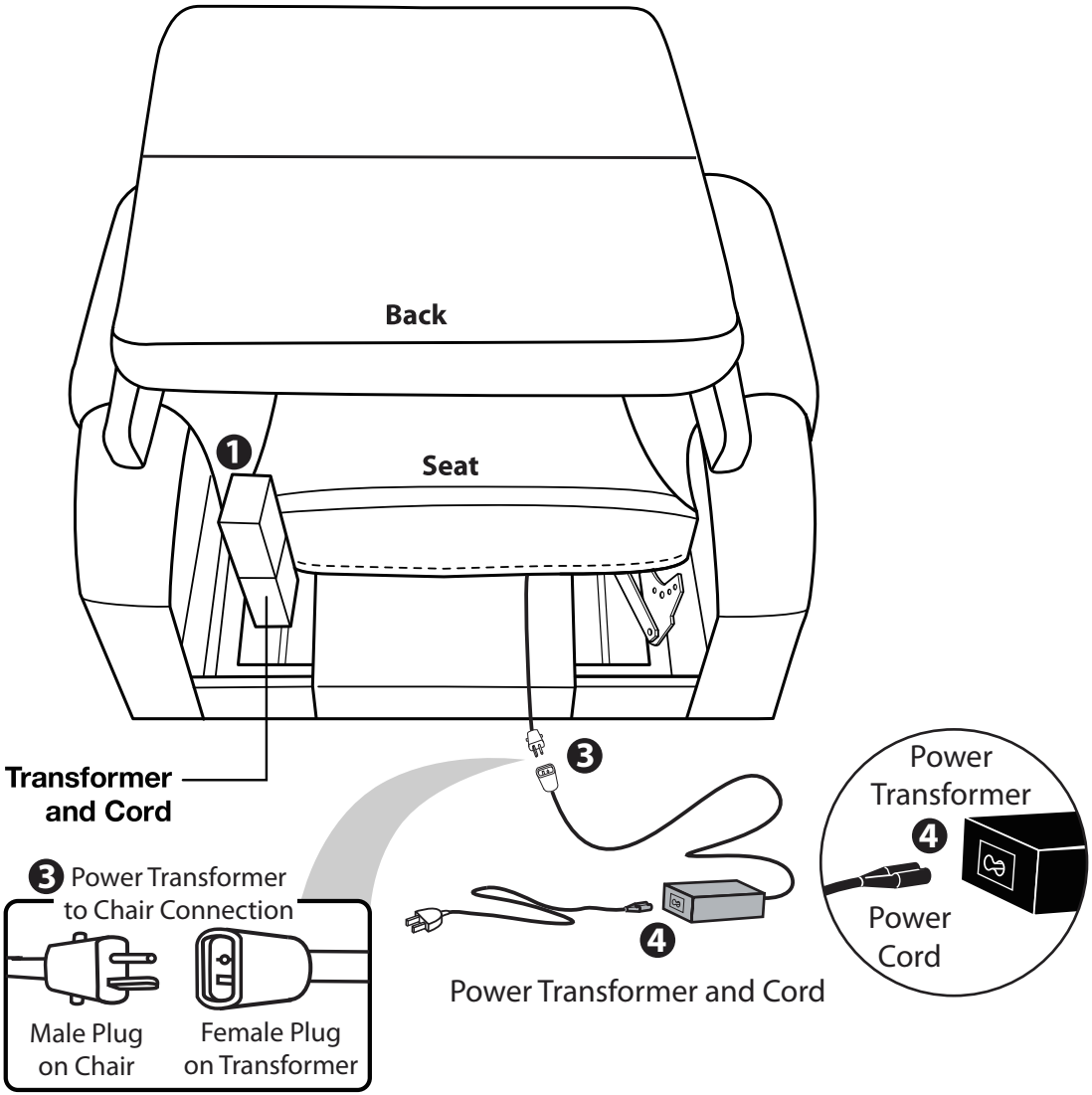

CHAIR ASSEMBLY

ATTACHING THE CHAIR BACK TO THE CHAIR BASE

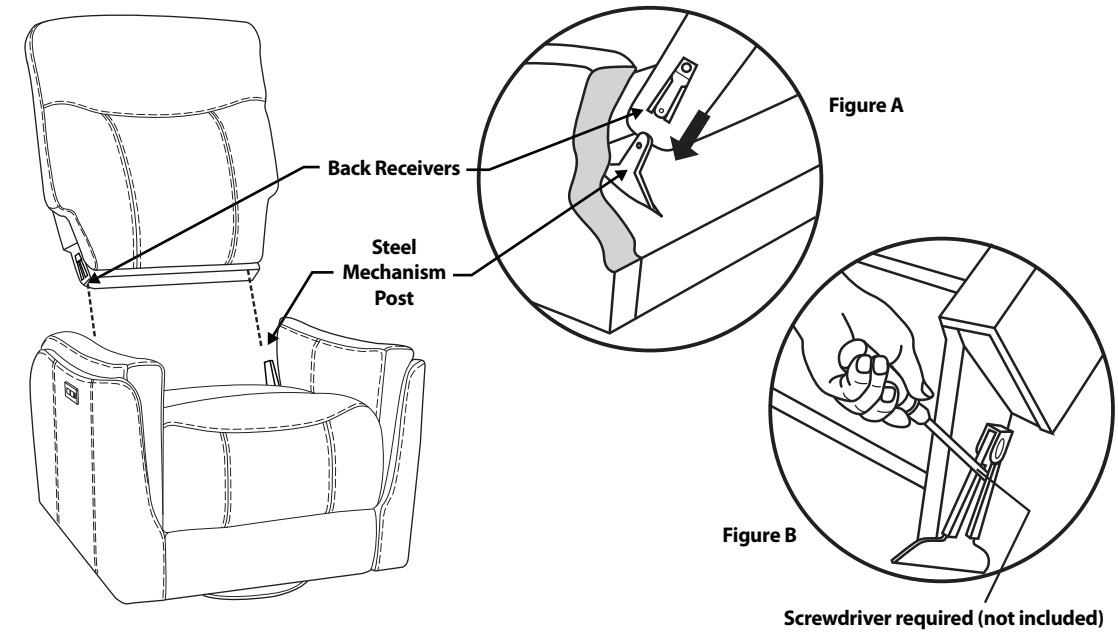

CAUTION: PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE STARTING ASSEMBLY PROCESS. IT IS RECOMMENDED TO USE TWO PEOPLE WHEN ASSEMBLING AND REMOVING THE INSIDE BACKS TO AND FROM THE BASE OF THE FURNITURE.

- STEP 1: Slide inside back receivers onto the steel mechanism posts in the base of the chair. (Figure A)
- STEP 2: Press both sides of the back firmly into place until the back automatically locks into place.
- STEP 3: To remove the back from the base, use a screwdriver pressing outward on the black tab in the metal receivers on both sides of the chair back and lift the back from the base of the chair.

ATTACHING THE CHAIR WINGS TO THE CHAIR BACK

