



ASSEMBLY INSTRUCTION

ITEM: RC622-PHLL4-082 Lyra Lift Chair w/PWR Recline/Headrest/Lumbar
RC622-PHLL4-091 Lyra Lift Chair w/PWR Recline/Headrest/Lumbar

Thank you for purchasing this quality product. Be sure to unpack carefully looking for small parts which may have come loose during shipment. Read all instruction and study the drawing carefully before starting the assembly process.

Inside the carton you will find two red plastic bags that have the recliner's wings in it. Remove the wings from the plastic bag and set them aside to be used in assembly and installation.

CAUTION: Make sure to read the instructions below before assembling your recliner. We suggest two people be used in the assembly of your recliner.

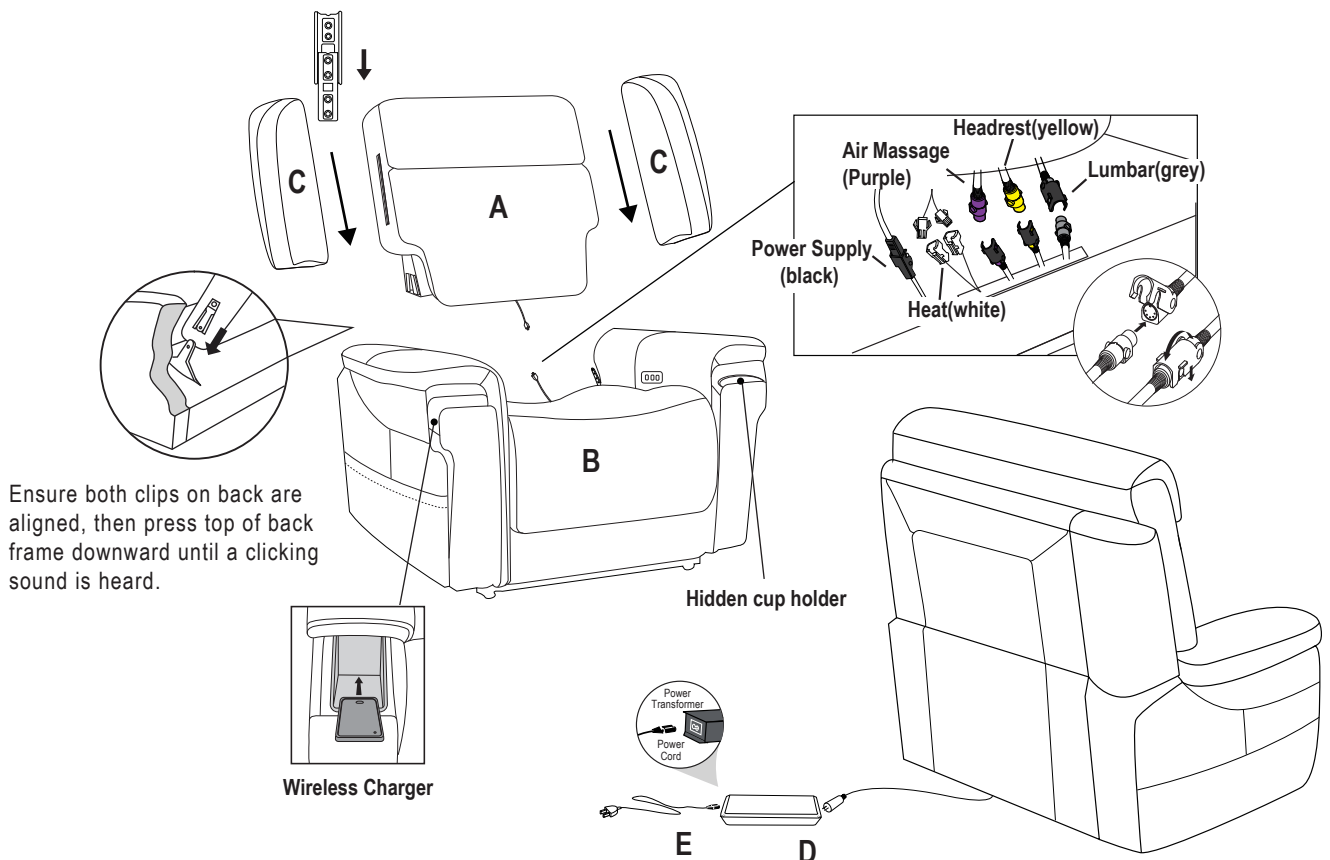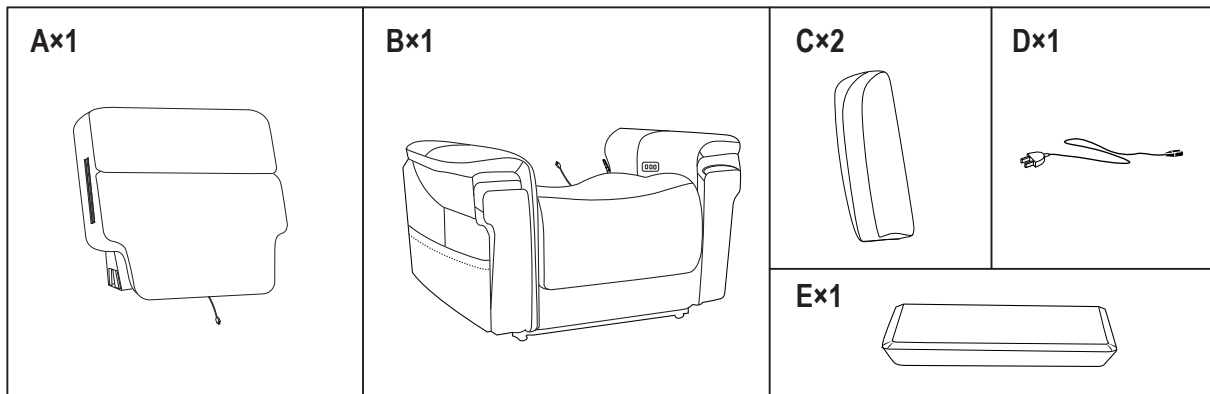

Do Not plug the power cord into a power outlet until the recliner is fully assembled and in place.

Notes:

- 1) To help prevent shock and or damage to your product only plug the power cord into a grounded outlet.
- 2) Read the operating instructions carefully before operating the recliner.

IMPORTANT NOTE


- To avoid any risk of suffocation to children or pets, dispose of all plastic bags immediately.
- To ensure proper performance, occasionally check for loose screws and tighten as needed.

OPERATING INSTRUCTION

ITEM: RC622-PHLL4-082 Lyra Lift Chair w/PWR Recline/Headrest/Lumbar
RC622-PHLL4-091 Lyra Lift Chair w/PWR Recline/Headrest/Lumbar

Thank you for purchasing this quality product. Be sure to unpack carefully looking for small parts which may have come loose during shipment. Read all instruction and study the drawing carefully before starting the assembly process.

CAUTION: Read the important instructions below before operating your power mechanized equipment
Do not allow children to play on or operate this power mechanized equipment

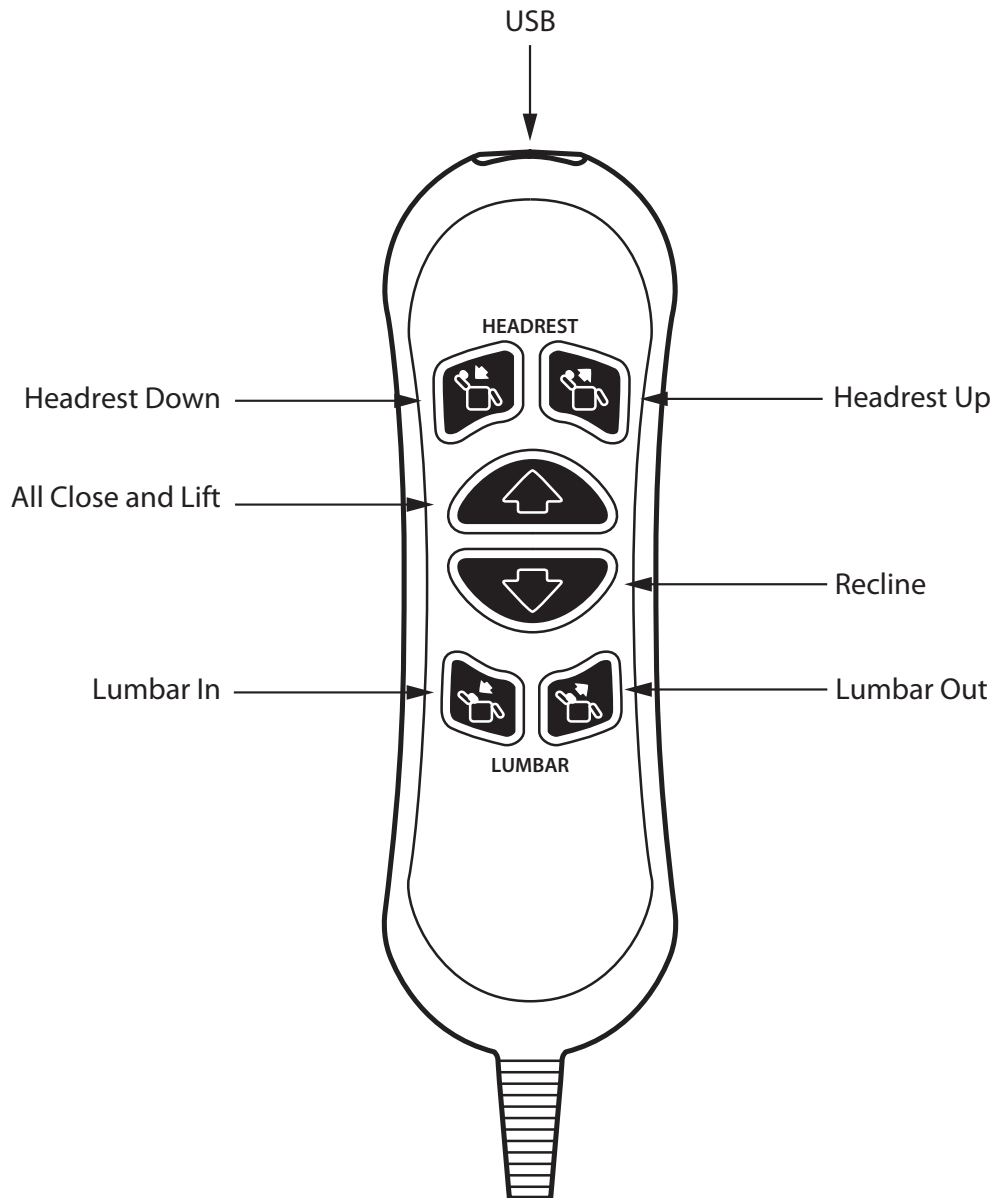


OPERATING INSTRUCTION

ITEM: RC622-PHLL4-082 Lyra Lift Chair w/PWR Recline/Headrest/Lumbar
RC622-PHLL4-091 Lyra Lift Chair w/PWR Recline/Headrest/Lumbar

Thank you for purchasing this quality product. Be sure to unpack carefully looking for small parts which may have come loose during shipment. Read all instruction and study the drawing carefully before starting the assembly process.

**CAUTION: Read the important instructions below before operating your power mechanized equipment
Do not allow children to play on or operate this power mechanized equipment**

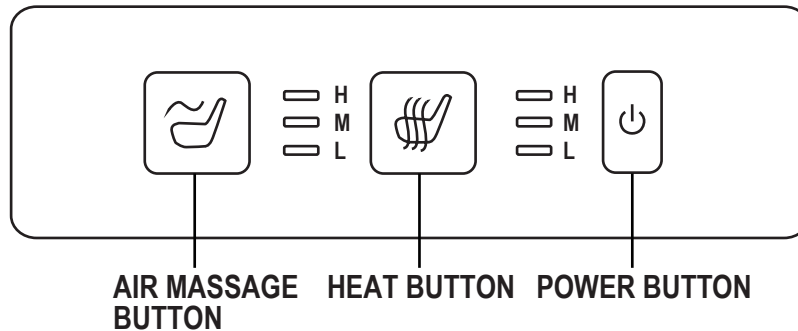

POWER BUTTON

Press the power button to turn the armrest control features on and off.

HEAT BUTTON

Press the heat button to activate the heat function. Press again to cycle intensity through low, medium, and high heat.

AIR MASSAGE BUTTON

Press the air massage button to activate the gentle pressure massage function. Press again to cycle through low, medium, and high intensity.

NOTE: This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device. Modifications not approved by the party responsible for compliance could void user's authority to operate the equipment.

Customer Service

If you have any questions regarding the use of this unit, or find that your unit needs service, please contact your retailer.

HOOKER®
FURNISHINGS